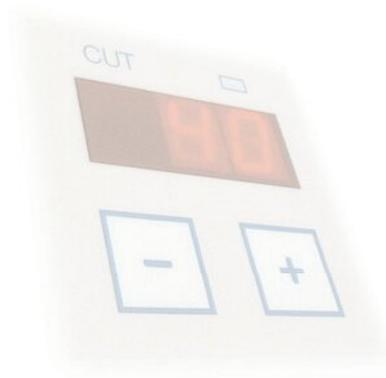
Aesculap Endoscopic Technology GN 300 Electrosurgical Unit

GN 300 Electrosurgical Unit

AESCULAP's electrosurgical unit GN 300 is designed to incorporate the principles of proven operating techniques and new requirements in the areas of Microsurgery, Neurosurgery and Minimally Invasive Surgery.

The microprocessor controlled GN 300 is easy to operate, and provides the surgeon ergonomic COAG control with a very high level of safety.

In conjunction with the corresponding software the microprocessor performs an automatic unit test after switching on and continuously monitors all signals relevant to a safe operation of the function keys and the connected accessories. Any irregularities are indicated to the user audibly and visually.

The multi-application adjustments, automatic coagulation and impedance-regulated power features make the GN 300 suitable for all monopolar and bipolar microsurgical and macro-surgical procedures in the fields of:

- General surgery
- Gynaecology
- Urology (including TUR)
- Orthopaedics
- ENT
- Neurosurgery
- Open-heart procedures

Compliance with national and international regulations is confirmed by test institutes.

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Memory

The individual indication-related empirical values can be stored in 3 permanent memory locations and rapidly called up by pressing the relevant key.

PDM

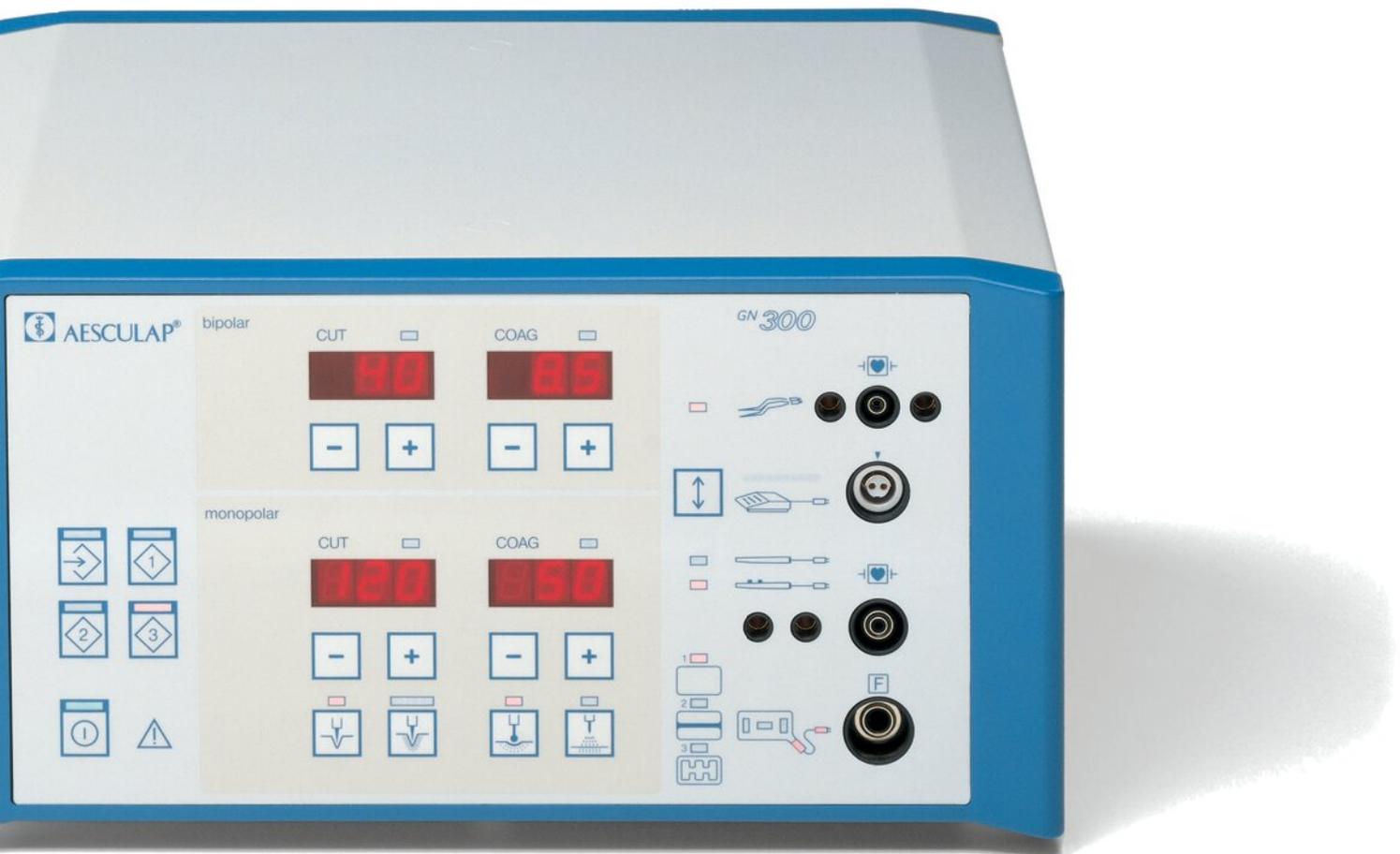
A permanent dynamic monitoring function monitors whether the split patient electrode is correctly positioned on the body. Monitoring is performed continuously also during HF output. An audible and visual warning is immediately given in critical situations.

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GN 300 Electrosurgical Unit

Monopolar CUT

A powerful sinus-type HF current with a low crest factor permits reproducible and consistent incision quality even with changing tissue characteristics. PSF (Power Start Function) ensures excision without difficulties, especially in endoscopic procedures. There are four levels of preselectable degrees of incision coagulation from which to choose.

Monopolar COAG

The power characteristics of the microprocessor controlled COAG mode are suitable for both large area and point coagulations, protecting tissue and allowing for considerable depth action without carbonization and adhesion. The Vario pedal controlled automatic coagulation feature provides added operative versatility.

Spray COAG

Spray COAG permits completely non-contact surface coagulation - without contact between electrodes and tissue.

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Bipolar CUT

In bipolar cutting, good cutting results can be achieved in conjunction with BiTOM instruments. Reduced wattage and a tightly restricted current flow zone result in practical advantages and increase patient safety.

Bipolar COAG

The wide field of bipolar coagulation is covered by an integrated MICRO/MACRO mode in conjunction with optimal adaptation to the tissue impedance. It is possible to perform very fine coagulations without tissue sticking of the forceps tips as well as extensive coagulations with corresponding instruments.

The Vario pedal controlled automatic coagulation feature provides added operative versatility.

Automatic coagulation

The output power of both monopolar and bipolar COAG modes is incrementally adjustable to the preselected setting via the Vario foot pedal control. The ergonomically designed COAG control provides the surgeon gentle coagulation without the risk of necrosis. Even microcoagulations can be performed sensitively and safely without changing the power setting. In addition, the post-operative healing phase is shortened and the life of the accessories increased.

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Refer to brochure C-304 for further accessories.

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GN300 Electrosurgical Unit mono- and bipolar, without mains cord and accessories

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GN209 Electrode handle with two finger push button

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GN203 Cable for electrode handles 3.5 m
GN205 5 m

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GK174R Surgical electrode set, complete, with 12 different electrodes Ø 4 mm, with container, rack and 12 electrodes

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GK107 Universal plate, single use 50 pieces in peel packs of 5 each, conductive area 110 cm²

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GN249 Cable for single use patient electrode 3.5 m
GN245 5 m

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Phone +49 7461 95-0
Fax +49 7461 95-2600

www.aesculap.de

All rights reserved. Technical alterations are possible. This leaflet may be used for no other purposes than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse we retain the rights to recall our catalogues and pricelists and to take legal actions.

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Aesculap AG & Co. KG