



The Winner of the
iF DESIGN AWARD 2016
Category Medicine / Healthcare



POWER SYSTEMS

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

As a surgeon you do the best job possible in order to restore your patients' quality of life, maintain their health or even save their lives, regardless of the constant pressure you are exposed to.

This responsibility made you a real perfectionist.

Peak performance, high precision and reliability are the claims you place upon yourself and anything that surrounds you. Because products that embody these values, support you in achieving your most ambitious goals.

Why should you accept compromises?



AT WORK



PRECISION



PERFORMANCE



PLEASURE

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Why did you choose the timepiece you are wearing? Because it is an artwork of accuracy, capturing value and eternal reliability? Do you appreciate how virtuously it is manufactured, even in the minutest detail? Did it become a part of you, hardly perceptible?

ELAN 4 aims to be the same – to meet the hugh demands in terms of precision, reliability and usability.

Will it convince you?



ERGONOMICS



INNOVATIVE GERMAN ENGINEERING:

- › New positioning of the motor before the angle, meaning there is no need for an angle gear or a motor coupling.

RESULT:

- › no couplings needed
- › handpieces become shorter and lighter
- › the angle does not get hot
- › sits comfortably in the hand
- › can be held more reliably and comfortably

BENEFITS:

*high-precision, convenient
and reliable grip*

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

What are the features you decide upon when buying a car?

Certainly you require reliability and enough horsepower. A car needs to accelerate quickly when required, it needs to cruise smoothly on long journeys and has to take you to your destination quickly and safely.

Power and reliability. This is what you can expect of ELAN 4, too. ELAN 4 is our answer to raise the bar in terms of overall performance.

Get the speed you need.



HIGH-SPEED



DESIGN
AWARD
2016

The Winner of the
iF DESIGN AWARD 2016
Category Medicine/Healthcare

INNOVATIVE GERMAN ENGINEERING:

- › integrated motors
- › on/off function integrated into the cable or hose to the motor coupling

RESULT:

The rotation speed is directly transferred, resulting in

- › very smooth running
- › no loss of power
- › high reliability
- › compact construction – no extra switch

BENEFITS:

quick, powerful and reliable work

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

In our high-tech world, simplicity is a luxury – “ease of use” and “joy of use” are almost synonyms. It’s more fun when things just work intuitively, without the need for headscratching or guesswork. Just like a smart phone: you simply need to follow your intuition. The apps you prefer are clearly arranged on the home screen. Personalized technology.

The same is true of the ELAN 4. You can select the motor drive type you are used to. The systems are straightforward and operation is self-explanatory. The couplings work according to the plug and play principle and the burrs fit every length of handpiece. Work will be easier and finally: even more joyful.

A new experience.



TWO SYSTEMS



INNOVATIVE GERMAN ENGINEERING:

- › all couplings are plug and play compatible
- › standardised burr lengths and clear coding of burrs
- › all operating elements are marked gold

RESULT:

- › easy to use
- › streamlined surgical set with fewer components
- › handpiece will always contain the correct motor
- › self-explanatory
- › all types and sizes of burrs are available for all handpiece lengths

BENEFITS:

The system is a joy to work with because it is set up clearly and simply. Fewer components and a universal burr length offer savings potentials.

AESCLAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

ELAN 4 electro



> GA800

ELAN 4 electro
control unit



> TE730 5 m > TE780 1.5 m

Power cord
Europe



> GA808

ELAN 4 electro
foot control



> GA806

GB073R

ELAN 4 electro
motor cable



> GA804

GB073R

ELAN 4 electro
motor cable with hand
switch



> GA805

GB073R

ELAN 4 electro
motor cable with hand
control



ELAN 4 – AN EXPERIENCE

Powerful and smooth-running, it operates with high precision. Thanks to its ergonomic design, it lies comfortably in your hand.



1/2

> GA822

GB076R

ELAN 4 electro perforator driver with Hudson coupling



1/2

> GA849

GB085R

ELAN 4 electro craniotome and multifunction handpiece (2-ring)



1/2

> GA824

GB077R

ELAN 4 electro low speed motor with Intra coupling

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

ELAN 4 electro



> GA861

GB083R

ELAN 4 electro
handpiece
standard
(1-ring) L4

> GA862

GB083R

ELAN 4 electro
handpiece
standard
(1-ring) L7

> GA863

GB083R

ELAN 4 electro
handpiece
standard
(1-ring) L10

> GA864

GB083R

ELAN 4 electro
handpiece
standard
(1-ring) L13



1/2

› GA865

GB083R

ELAN 4 electro
handpiece
heavy duty
(2-ring) L7



1/2

› GA866

GB083R

ELAN 4 electro
handpiece
heavy duty
(2-ring) L10



1/2

› GA867

GB083R

ELAN 4 electro
handpiece
heavy duty
(2-ring) L13



1/2

› GA868

GB083R

ELAN 4 electro
handpiece
heavy duty
(2-ring) L17



1/2

› GA869

GB083R

ELAN 4 electro
handpiece
heavy duty
(2-ring) L22

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

ELAN 4 electro



1/2

> GA860

GB084R

ELAN 4 electro
MIS handpiece



1/2

> GB920R

GB723R

ELAN 4
MIS handpiece
shaft, straight
L10



1/2

> GB921R

GB723R

ELAN 4
MIS handpiece
shaft, curved
L10



1/2

> GB925R

GB723R

ELAN 4
MIS handpiece
shaft, straight
L13



1/2

> GB926R

GB723R

ELAN 4
MIS handpiece
shaft, curved
L13



› GA836

GB086R

ELAN 4 electro
micro sagittal saw



› GA837

GB087R

ELAN 4 electro
micro reciprocating saw



› GA831

GB088R

ELAN 4 electro
sagittal saw



› GA832

GB087R

ELAN 4 electro
reciprocating saw

AESFULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

ELAN 4 air



- > **GA710R** AESFULAP® Draeger
- > **GA712R** Schrader
- > **GA713R** DIN
- > **GA715R** Synthes

ELAN 4 air
wall adapter



- > **GA702R** 3 m
 - > **GA703R** 5 m
- ELAN 4 air
wall hose



- > **GA708**
- ELAN 4 air
foot control



- > **GA705R**
GB053R
- ELAN 4 air
motor hose 3 m



- > **GA706R** 3 m
 - > **GA707R** 5 m
- GB053R**
- ELAN 4 air
motor hose
with hand control





ELAN 4 – SIMPLE HANDLING

The clear and simple system set-up is a joy to work with. Every handpiece always contains the correct motor, reducing the risk of errors.



1/2



1/2



1/2

> GA722

GB056R

ELAN 4 air
perforator driver
with Hudson
coupling

> GA749

GB065R

ELAN 4 air
craniotome and
multifunction
handpiece (2-ring)

> GA724

GB057R

ELAN 4 air
low speed motor
with Intra coupling

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

ELAN 4 air



> GA761

GB063R

ELAN 4 air
handpiece
standard
(1-ring) L4

> GA762

GB063R

ELAN 4 air
handpiece
standard
(1-ring) L7

> GA763

GB063R

ELAN 4 air
handpiece
standard
(1-ring) L10

> GA764

GB063R

ELAN 4 air
handpiece
standard
(1-ring) L13



› GA765

GB063R

ELAN 4 air
handpiece
heavy duty
(2-ring) L7



› GA766

GB063R

ELAN 4 air
handpiece
heavy duty
(2-ring) L10



› GA767

GB063R

ELAN 4 air
handpiece
heavy duty
(2-ring) L13



› GA768

GB063R

ELAN 4 air
handpiece
heavy duty
(2-ring) L17



› GA769

GB063R

ELAN 4 air
handpiece
heavy duty
(2-ring) L22

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

ELAN 4 air



1/2

> GA760

GB064R

ELAN 4 air
MIS handpiece



1/2

> GB920R

GB723R

ELAN 4
MIS handpiece
shaft, straight
L10



1/2

> GB921R

GB723R

ELAN 4
MIS handpiece
shaft, curved
L10



1/2

> GB925R

GB723R

ELAN 4
MIS handpiece
shaft, straight
L13



1/2

> GB926R

GB723R

ELAN 4
MIS handpiece
shaft, curved
L13



> GA736

GB066R

ELAN 4 air
micro sagittal saw



> GA737

GB067R

ELAN 4 air
micro reciprocating saw



> GA731

GB068R

ELAN 4 air
sagittal saw



> GA732

GB067R

ELAN 4 air
reciprocating saw

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

ACCESSORIES



› GB941R

GB719R


ELAN 4
fixed dura guard
pediatric



› GB942R

GB719R

ELAN 4
fixed dura guard
standard

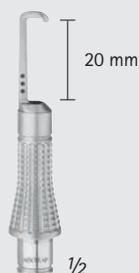

› GB943R

GB719R

ELAN 4
fixed dura guard
long



› GB945R

GB719R

ELAN 4
holding sleeve for
craniotome and
multifunction
handpiece



› GB947R

GB719R

ELAN 4
turnable dura guard
standard



ELAN 4 FEATURES ECONOMIC ADVANTAGES

The streamlined surgical set and the universal burr length help reducing inventory. As a result, there is a considerable saving potential.



› GA395SU



ELAN 4 electro disposable tube set for irrigation pump



› Disposable spray nozzles for ELAN 4

- GA258SU ELAN 4 spray nozzle for saws
- GA259SU ELAN 4 spray nozzle for cranitome attachments
- GA261SU ELAN 4 spray nozzle for 1-ring handpiece L4
- GA262SU ELAN 4 spray nozzle for 1-ring handpiece L7
- GA263SU ELAN 4 spray nozzle for 1-ring handpiece L10
- GA264SU ELAN 4 spray nozzle for 1-ring handpiece L13
- GA265SU ELAN 4 spray nozzle for 2-ring handpiece L7
- GA266SU ELAN 4 spray nozzle for 2-ring handpiece L10
- GA267SU ELAN 4 spray nozzle for 2-ring handpiece L13
- GA268SU ELAN 4 spray nozzle for 2-ring handpiece L17
- GA269SU ELAN 4 spray nozzle for 2-ring handpiece L22



› GB866R


ELAN 4 lateral burr guard for 1-ring handpiece and a maximum burr diameter of 4.5 mm

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

REPROCESSING

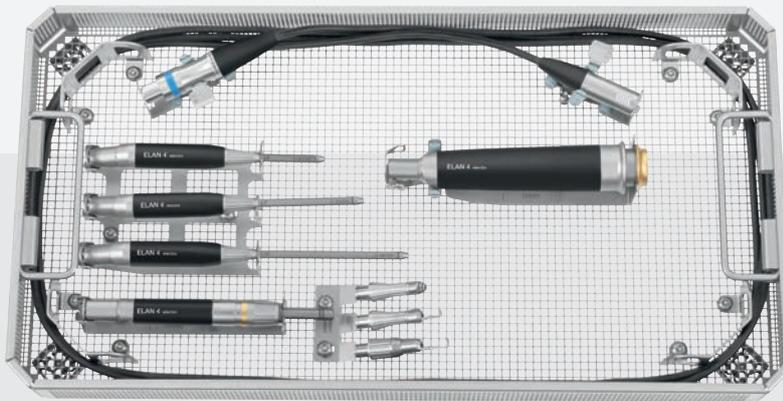

ECCOS set ELAN 4 electro GB261R

› ELAN 4 electro ECCOS holder

GB261R	ECCOS set ELAN 4 electro
GB072R*	ECCOS cable/hose positioning guide
GB073R*	ECCOS holder for motor cable
GB076R*	ECCOS holder for perforator driver GA822
GB077R	ECCOS holder for low speed motor Intra GA824
GB082R	ECCOS holder for 1 ELAN 4 handpiece
GB083R*	ECCOS holder for 3 ELAN 4 handpieces
GB084R	ECCOS holder for MIS handpiece GA860
GB723R	ECCOS holder for 2 MIS shafts
GB085R*	ECCOS holder for craniotome GA849
GB086R	ECCOS holder for micro sagittal saw GA836
GB087R	ECCOS holder for reciprocating saws GA832, GA837
GB088R	ECCOS holder for sagittal saw GA831
GB572R	ECCOS holder for cranial perforators GB300R-GB304R
GB718R	ECCOS holder for 12 burs
GB719R*	ECCOS holder for 3 craniotome attachments
GB720R	ECCOS universal holder for saw blades
GB729R	ECCOS 2 for reciprocating saw shafts

› ELAN 4 air ECCOS holder

GB260R	ECCOS set ELAN 4 air
GB072R*	ECCOS cable/hose positioning guide
GB053R*	ECCOS holder for motor hose
GB056R*	ECCOS holder for perforator driver GA722
GB057R	ECCOS holder for low speed motor Intra GA724
GB062R	ECCOS holder for 1 ELAN 4 handpiece
GB063R*	ECCOS holder for 3 ELAN 4 handpieces
GB064R	ECCOS holder for MIS handpiece GA760
GB723R	ECCOS holder for 2 MIS shafts
GB065R*	ECCOS holder for craniotome GA749
GB066R	ECCOS holder for micro sagittal saw GA736
GB067R	ECCOS holder for micro reciprocating saws GA732, GA737
GB068R	ECCOS holder for sagittal saw GA731
GB572R	ECCOS holder for cranial perforators GB300R-GB304R
GB718R	ECCOS holder for 12 burs
GB719R*	ECCOS holder for 3 craniotome attachments
GB720R	ECCOS universal holder for saw blades
GB729R	ECCOS 2 for reciprocating saw shafts

ELAN 4 – MODERN HYGIENE MANAGEMENT

ELAN 4 meets the increasingly strict requirements and sets standards for the sterile processing of motor systems.



The cleaning device allows reliable inner cleaning of the applied part.



› ELAN 4 electro rinsing device + accessories

- GB692R ELAN 4 electro rinsing device
- GB600860 ELAN 4 electro oil spray adaptor
- GB679R ELAN 4 rinsing device for 4 MIS shafts
- GB600870 ELAN 4 MIS oil spray adaptor
- GB600 STERILIT Power Systems oil spray
- GB693 Ceramic filter for rinsing device

› ELAN 4 air rinsing device + accessories

- GB691R ELAN 4 air rinsing device
- GB600850 ELAN 4 air oil spray adaptor
- GB679R ELAN 4 rinsing device for 4 MIS shafts
- GB600870 ELAN 4 MIS oil spray adaptor
- GB600 STERILIT Power Systems oil spray
- GB693 Ceramic filter for rinsing device

AESCULAP® ELAN 4


PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

BLADES

SAW BLADES – SAGITTAL SAW  sterile

for GA736 and GA836

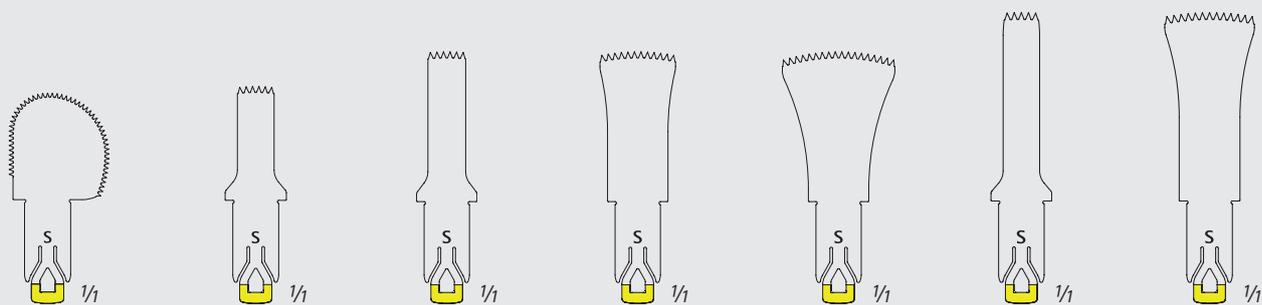

> GP491R

Saw blade
S sagittal
14/13/0.3 mm

> GP492R

Saw blade
S sagittal
15/5/0.3 mm a

> GP493R

Saw blade
S sagittal
20/5/0.3 mm

> GP494R

Saw blade
S sagittal
20/10/0.3 mm

> GP495R

Saw blade
S sagittal
20/15/0.3 mm

> GP496R

Saw blade
S sagittal
25/5/0.3 mm

> GP497R

Saw blade
S sagittal
25/12/0.3 mm




ELAN 4 – ENGINEERING EXPERTISE

ELAN 4 is produced using state-of-the-art manufacturing technology in Tuttlingen, Germany, at the Aesculap AG Innovation Factory, which was specifically built to produce motor and sterile container systems.

SAW BLADES – SAGITTAL SAW sterile

for GA731 and GA831

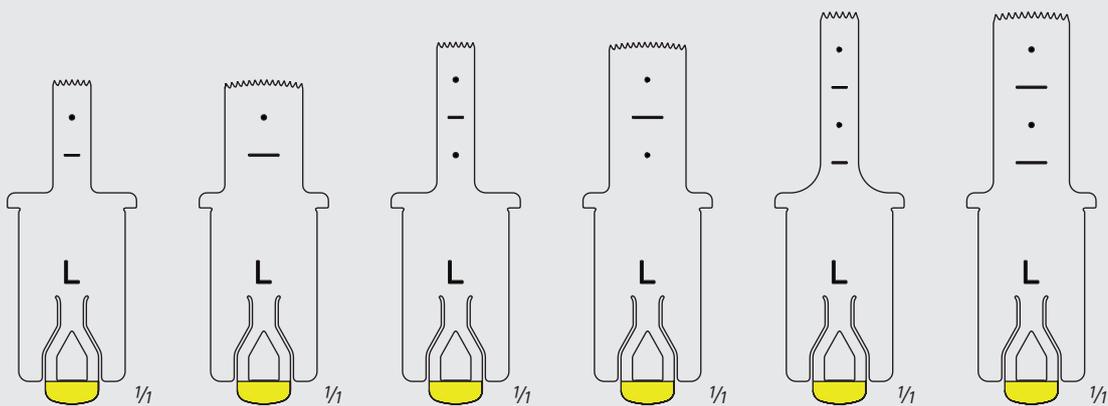

> GP501R

Saw blade
L sagittal
15/5/0.5 mm

> GP502R

Saw blade
L sagittal
15/10/0.5 mm

> GP504R

Saw blade
L sagittal
20/5/0.5 mm

> GP505R

Saw blade
L sagittal
20/10/0.5 mm

> GP514R

Saw blade
L sagittal
25/5/0.7 mm

> GP515R


Saw blade
L sagittal
25/10/0.7 mm

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

BLADES

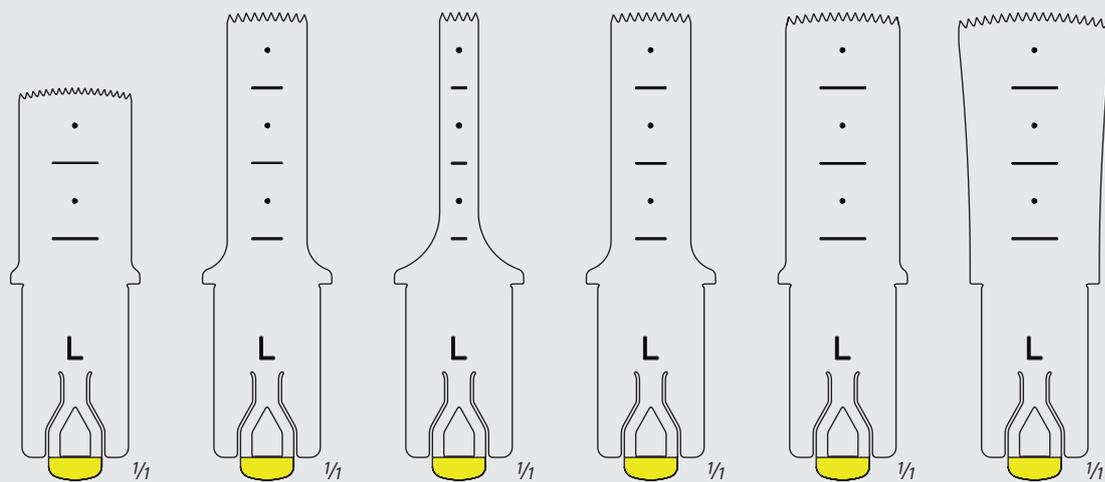

> GP516R

Saw blade
L sagittal
25/15/0.7 mm

> GP518R

Saw blade
L sagittal
35/10/0.5 mm

> GP521R

Saw blade
L sagittal
35/5/0.5 mm

> GP522R


Saw blade
L sagittal
35/10/0.7 mm

> GP523R

Saw blade
L sagittal
35/15/0.7 mm

> GP524R

Saw blade
L sagittal
35/20/0.7 mm

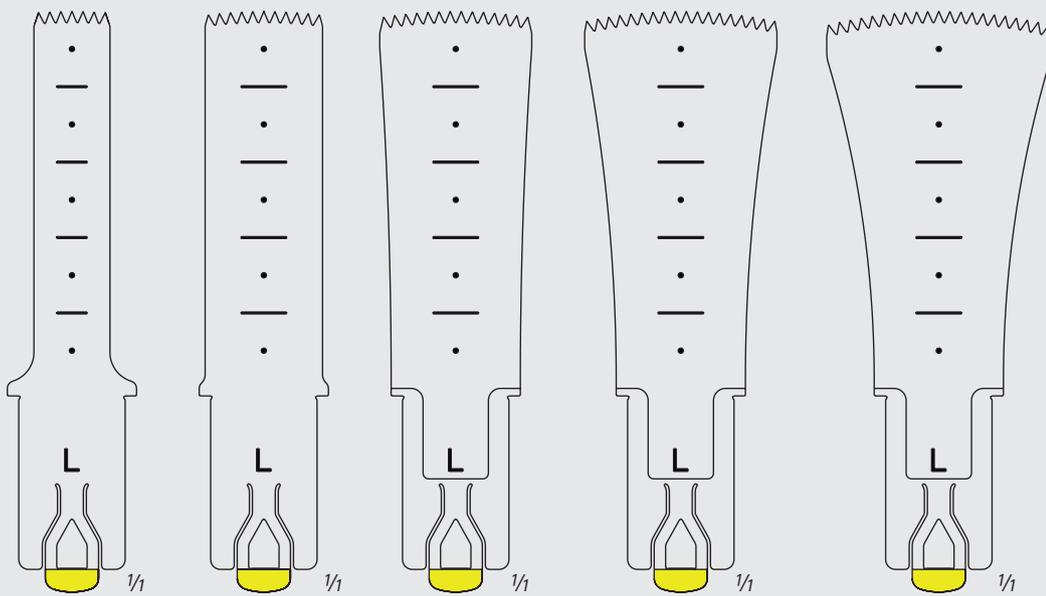

➤ **GP535R**

Saw blade
L sagittal
50/10/0.8 mm

➤ **GP536R**

Saw blade
L sagittal
50/15/0.8 mm

➤ **GP537R**

Saw blade
L sagittal
50/20/0.9 mm

➤ **GP538R**

Saw blade
L sagittal
50/25/0.9 mm

➤ **GP539R**


Saw blade
L sagittal
50/30/0.9 mm

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.


Engineered in Germany.

BLADES

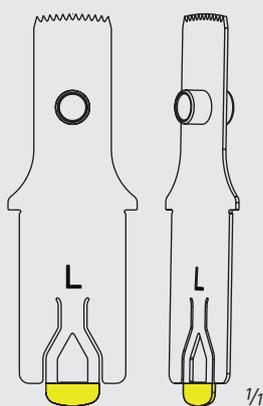

> GP511R

Saw blade
with depth
10/10/0.5 mm

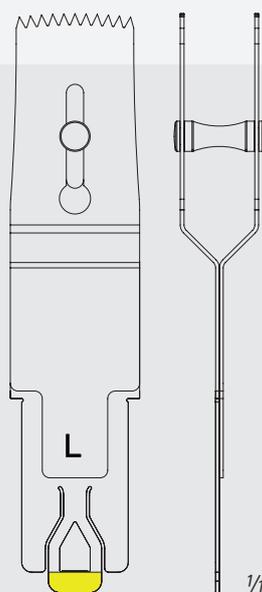


> GP526R-GP532R

Saw blades
L sagittal
20/15 mm

GP526R	6 mm
GP527R	7 mm
GP528R	8 mm
GP529R	9 mm
GP530R	10 mm
GP531R	11 mm
GP532R	12 mm

SAW BLADES – RECIPROCATING SAW sterile

for GA737 and GA837

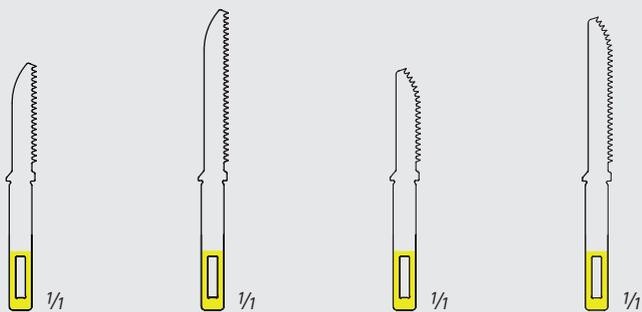

› GP542R

Reciprocating
saw blade
micro
13/0.3 mm

› GP543R

Reciprocating
saw blade
micro
20/0.3 mm

› GP544R

Reciprocating
saw blade
micro
13/0.3 mm

› GP545R

Reciprocating
saw blade
micro
20/0.3 mm

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

BLADES

SAW BLADES – RECIPROCATING SAW sterile


for GA732 and GA832



> GP551R, GP561R

Reciprocating saw blades

GP551R 25/0.6 mm
GP561R 25/0.6 mm,
long

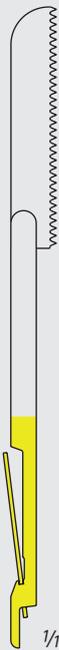

> GP552R, GP553R

Reciprocating saw blades

GP552R 27/0.35 mm
GP553R 27/0.6 mm



> GP562R, GP563R

Reciprocating saw blades


GP562R 27/0.35 mm,
long
GP563R 27/0.6 mm,
long



> GP554R, GP564R

Reciprocating saw blades

GP554R 33/0.6 mm
GP564R 33/0.6 mm,
long

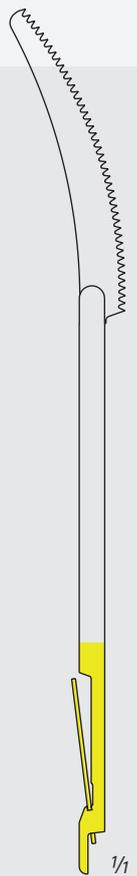

> GP565R

Banana saw blade long

37/0.6 mm



> GP550R

Reciprocating saw blade
15/0.6 mm



> GP575R

Reciprocating saw blade
conv.
17/0.6 mm



> GP576R

Reciprocating saw blade
conv.
24/0.6 mm



> GP559R

Shank for reciprocating saw blade
GC615R/GC633R


sterile



> GP569R


Shank long for reciprocating saw blade
GC615R/GC633R

sterile

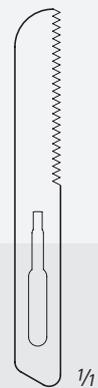

> GC615R

reciprocating saw blade
23/0.25 mm

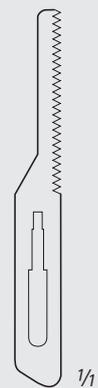

> GC633R


reciprocating saw blade
23/0.25 mm

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

BLADES

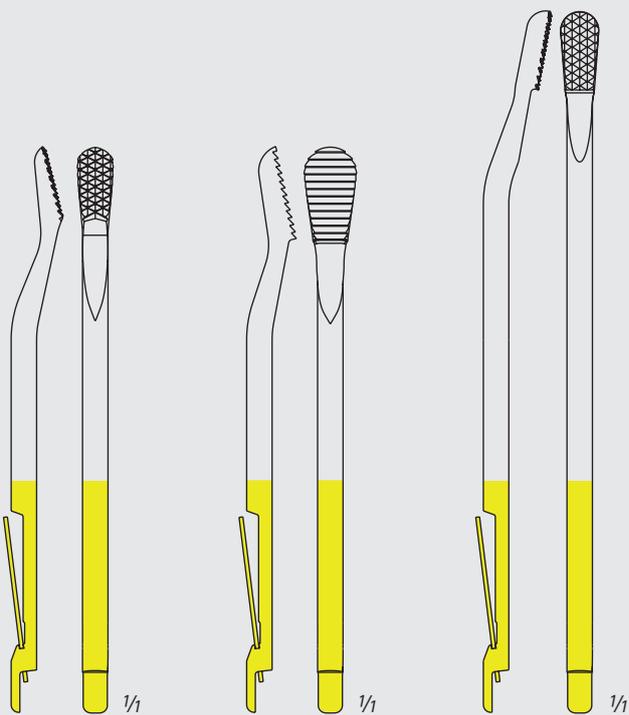

> GP557R

Bone rasp for reciprocating saw narrow

> GP558R

Bone rasp for reciprocating saw wide

> GP567R

Bone rasp for reciprocating saw narrow, long



➤ **GB718R**

ECCOS
holder for
12 burs



➤ **GB720R**

ECCOS
universal holder for
saw blades



➤ **GB729R**

ECCOS
holder for
2 reciprocating
saw shafts

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

BURRS



BURRS FOR 1-RING HANDPIECES

> Rosen burrs

	0.6 mm	0.8 mm	1.0 mm	1.0 mm	1.4 mm	1.4 mm	1.8 mm	1.8 mm	2.3 mm	2.3 mm
Art No.	GP111R*	GP112R*	GP113R	GP114R*	GP115R	GP116R*	GP117R	GP118R*	GP120R	GP121R*
	2.7 mm	2.7 mm	3.0 mm	3.5 mm	4.0 mm	4.5 mm	5.0 mm	6.0 mm		
Art No.	GP122R	GP123R*	GP124R	GP125R	GP126R	GP127R	GP128R	GP129R		

> Rosen burrs, soft cut

	3.0 mm	4.0 mm	5.0 mm	6.0 mm
Art No.	GP133R*	GP134R	GP135R	GP136R

> Twin-Cut burrs

	4.0 mm	5.0 mm	6.0 mm	7.0 mm
Art No.	GP184R	GP185R	GP186R	GP187R



BURRS FOR 1-RING HANDPIECES

> Diamond burrs

	0.6 mm	0.8 mm	1.0 mm	1.0 mm	1.4 mm	1.4 mm	1.8 mm	1.8 mm	2.3 mm	2.3 mm
Art No.	GP141R*	GP142R*	GP143R	GP144R*	GP145R	GP146R*	GP147R	GP148R*	GP149R	GP150R*
	2.7 mm	2.7 mm	3.0 mm	3.5 mm	4.0 mm	4.5 mm	5.0 mm	6.0 mm		
Art No.	GP151R	GP152R*	GP153R	GP154R	GP155R	GP156R	GP157R	GP158R		

> Diamond burrs, coarse

	2.0 mm	2.0 mm	2.3 mm	2.3 mm	3.0 mm	3.0 mm	4.0 mm	5.0 mm	6.0 mm
Art No.	GP161R	GP162R*	GP163R	GP164R*	GP165R	GP166R*	GP168R	GP169R	GP170R

> Diamond burrs, extra coarse

	3.0 mm	4.0 mm	5.0 mm	6.0 mm	7.0 mm
Art No.	GP173R	GP174R	GP175R	GP176R	GP177R

* Burrs with longer overhang

AESFULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

BURRS



BURRS FOR 1-RING HANDPIECES

> Neuro cutter, US-Type

	1.5 mm	2.0 mm	2.5 mm	3.0 mm	3.0 mm
Art No.	GP201R*	GP202R*	GP203R*	GP204R	GP205R*

> Neuro cutter, AESFULAP®-Type

	2.0 mm	2.5 mm	3.0 mm
Art No.	GP208R	GP209R	GP210R

> Neuro cutter, diamond coarse

	1.5 mm	1.5 mm	2.0 mm	2.0 mm	2.5 mm	2.5 mm	3.0 mm	3.0 mm	4.0 mm	4.0 mm
Art No.	GP211R	GP212R*	GP213R	GP214R*	GP215R	GP216R*	GP217R	GP218R*	GP219R	GP220R*

> Barrel burrs, standard

	4.0 mm	5.0 mm	6.0 mm
Art No.	GP224R*	GP225R	GP226R

> Barrel burrs, soft cut

	4.0 mm	5.0 mm	6.0 mm
Art No.	GP228R*	GP229R	GP230R

> Cone burrs

	4.0 mm	5.0 mm	6.0 mm
Art No.	GP234R	GP235R	GP236R

> Reverse Taper burr, diamond coarse

	4.0 mm
Art No.	GP238R*

BURRS FOR 1-RING HANDPIECES

> Acorn burrs

	3.0 mm	4.0 mm	5.0 mm	6.0 mm
Art No.	GP193R	GP194R	GP195R	GP196R

> Oval burrs

	4.0 mm	5.5 mm
Art No.	GP232R	GP233R

> Lindemann

	1.8 mm	2.1 mm	2.3 mm	2.3 mm
Length	7 mm	10 mm	10 mm	22 mm
Art No.	GP241R	GP242R	GP243R	GP240R

> Twist drills

	1.5 mm	2.0 mm
Art No.	GP188R	GP189R

> Pin cutter

	1.0 mm
Art No.	GP190R

> Tungsten carbide-Rosen burrs

	1.0 mm	2.0 mm	3.0 mm	4.0 mm	5.0 mm	6.0 mm
Art No.	GP113TC	GP119TC	GP124TC	GP126TC	GP128TC	GP129TC

> Tungsten carbide-Neuro cutter

	3.0 mm
Art No.	GP210TC

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

BURRS



BURRS FOR 2-RING HANDPIECES

› Rosen burrs

		2.3 mm	3.0 mm	3.5 mm	4.0 mm	4.5 mm	5.0 mm	6.0 mm	7.0 mm	8.0 mm
Art No.	GP301R	GP302R	GP303R	GP304R	GP305R	GP306R	GP307R	GP308R	GP309R	

› Diamond burrs

		2.3 mm	3.0 mm	3.5 mm	4.0 mm	4.5 mm	5.0 mm	6.0 mm		
Art No.	GP311R	GP312R	GP313R	GP314R	GP315R	GP316R	GP317R			

› Diamond burrs, coarse

		2.3 mm	3.0 mm	4.0 mm	5.0 mm	6.0 mm				
Art No.	GP321R	GP322R	GP323R	GP324R	GP325R					

› Diamond burrs, extra coarse

		4.0 mm	5.0 mm	6.0 mm						
Art No.	GP328R	GP329R	GP330R							



BURRS FOR 2-RING HANDPIECES

> Neuro cutter, US-type

	2.0 mm	2.5 mm	3.0 mm	3.0 mm
Art No.	GP361R	GP362R	GP363R	GP364R*

> Neuro cutter, AESCULAP®-type

	3.0 mm
Art No.	GP367R*

> Neuro cutter, diamond coarse

	3.0 mm	3.0 mm
Art No.	GP370R	GP371R*

> Barrel burrs, standard

	4.0 mm	5.0 mm	6.0 mm
Art No.	GP374R*	GP375R	GP376R

> Barrel burrs, soft cut

	4.0 mm	5.0 mm	6.0 mm
Art No.	GP378R*	GP379R*	GP380R

> Cone burrs

	4.0 mm	5.0 mm	6.0 mm
Art No.	GP384R	GP385R	GP386R

> Reverse Taper burrs, diamond coarse

	4.0 mm	6.0 mm
Art No.	GP388R*	GP389R*

* Burrs with longer overhang

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

BURRS



BURRS FOR 2-RING HANDPIECES

> Acorn burrs

	5.0 mm	6.0 mm	7.5 mm	9.0 mm
Art No.	GP355R	GP356R	GP357R	GP358R


> Oval burr

	4.0 mm
Art No.	GP382R

> Lindemann

	1.8 mm	2.1 mm	2.3 mm	2.3 mm
Length	7 mm	10 mm	10 mm	22 mm
Art No.	GP391R	GP392R	GP393R	GP397R

> Pin cutter

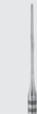	1.0 mm
Art No.	GP350R

> Craniotome cutter, spiral type

	Length	•	••	•••
Art No.	GP341R	GP342R	GP343R	

> Craniotome cutter, straight type

	Length	•	••	•••
Art No.	GP351R	GP352R	GP353R	

> Twin-Cut burrs

	5.0 mm	6.0 mm	7.0 mm	8.0 mm	9.0 mm
Art No.	GP335R	GP336R	GP337R	GP338R	GP339R

> Diamond disc ②

	25 mm
Art No.	GP398SU

BURRS FOR 2-RING HANDPIECES

› Twist drills (cutting depth 4 mm – for pilot holes)

	1.0 mm	1.1 mm	1.2 mm	1.5 mm
Art No.	GP344R	GP345R	GP346R	GP347R

› Twist drills

	1.5 mm	2.0 mm
Art No.	GP348R	GP349R

› Tungsten carbide-Rosen burr

	2.3 mm	3.0 mm	4.0 mm	5.0 mm	6.0 mm	7.0 mm	8.0 mm
Art No.	GP301TC	GP302TC	GP304TC	GP306TC	GP307TC	GP308TC	GP309TC

› Tungsten carbide-Neuro cutter

	3.0 mm
Art No.	GP367TC

› Tungsten carbide-Side cutter

	3.0 mm
Art No.	GP399TC-SU

› Tungsten carbide-Taper burr

	1.2 mm	1.6 mm
Art No.	GP395TC	GP396TC

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

BURRS



BURRS FOR MIS HANDPIECES

> Rosen burrs

	3.0 mm	4.0 mm
L10	GP422SU	GP423SU
L13	GP442SU	GP443SU

> Neurocutter

	2.0 mm	3.0 mm
L10	GP420SU	GP421SU
L13	GP440SU	GP441SU

> Diamond burrs

	3.0 mm	4.0 mm	5.0 mm
L10	GP432SU	GP433SU	GP434SU
L13	GP452SU	GP453SU	GP454SU

> Diamond burrs, extra coarse

	3.0 mm	4.0 mm	5.0 mm
L10	GP435SU	GP436SU	GP437SU
L13	GP455SU	GP456SU	GP457SU

BURRS FOR MIS HANDPIECES

> Neurocutter, diamond

	2.0 mm	3.0 mm	4.0 mm	5.0 mm
 L10	GP424SU	GP425SU	GP426SU	GP427SU
L13	GP444SU	GP445SU	GP446SU	GP447SU

> Neurocutter, diamond extra coarse

	3.0 mm	4.0 mm	5.0 mm
 L10	GP428SU	GP429SU	GP430SU
L13	GP448SU	GP449SU	GP450SU

A series of horizontal dotted lines for writing, spanning the width of the page.

AESFULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

ORDER INFORMATION

> ELAN 4 electro

GA800	ELAN 4 electro control unit	GA863	ELAN 4 electro handpiece standard (1-ring) L10
TE730	Power cord Europe length: 5 m	GA864	ELAN 4 electro handpiece standard (1-ring) L13
TE780	Power cord Europe length: 1.5 m	GA865	ELAN 4 electro handpiece heavy duty (2-ring) L7
GA808	ELAN 4 electro foot control	GA866	ELAN 4 electro handpiece heavy duty (2-ring) L10
GA806	ELAN 4 electro motor cable	GA867	ELAN 4 electro handpiece heavy duty (2-ring) L13
GA804	ELAN 4 electro motor cable with hand switch	GA868	ELAN 4 electro handpiece heavy duty (2-ring) L17
GA805	ELAN 4 electro motor cable with hand control	GA869	ELAN 4 electro handpiece heavy duty (2-ring) L22
GA822	ELAN 4 electro perforator driver with Hudson coupling	GA860	ELAN 4 electro MIS handpiece
GA849	ELAN 4 electro craniotome and multifunction handpiece (2-ring)	GA836	ELAN 4 electro micro sagittal saw
GA824	ELAN 4 electro low speed motor with Intra coupling	GA837	ELAN 4 electro micro reciprocating saw
GA861	ELAN 4 electro handpiece standard (1-ring) L4	GA831	ELAN 4 electro sagittal saw
GA862	ELAN 4 electro handpiece standard (1-ring) L7	GA832	ELAN 4 electro reciprocating saw

> ELAN 4 air

GA710R	ELAN 4 air wall-adaptor AESFULAP®-Draeger	GA762	ELAN 4 air handpiece standard (1-ring) L7
GA712R	ELAN 4 air wall-adaptor Schrader with diffusor	GA763	ELAN 4 air handpiece standard (1-ring) L10
GA713R	ELAN 4 air wall-adaptor DIN with diffusor	GA764	ELAN 4 air handpiece standard (1-ring) L13
GA715R	ELAN 4 air wall-adaptor Synthes	GA765	ELAN 4 air handpiece heavy duty (2-ring) L7
GA702R	ELAN 4 air wall hose 3 m	GA766	ELAN 4 air handpiece heavy duty (2-ring) L10
GA703R	ELAN 4 air wall hose 5 m	GA767	ELAN 4 air handpiece heavy duty (2-ring) L13
GA708	ELAN 4 air foot control	GA768	ELAN 4 air handpiece heavy duty (2-ring) L17
GA705R	ELAN 4 air motor hose 3 m	GA769	ELAN 4 air handpiece heavy duty (2-ring) L22
GA706R	ELAN 4 air motor hose 3 m with hand control	GA760	ELAN 4 air MIS handpiece
GA707R	ELAN 4 air motor hose 5 m with hand control	GA736	ELAN 4 air micro sagittal saw
GA722	ELAN 4 air perforator driver with Hudson coupling	GA737	ELAN 4 air micro reciprocating saw
GA749	ELAN 4 air craniotome and multifunction handpiece (2-ring)	GA731	ELAN 4 air sagittal saw
GA724	ELAN 4 air low speed motor with Intra coupling	GA732	ELAN 4 air reciprocating saw
GA761	ELAN 4 air handpiece standard (1-ring) L4		

> Attachments for craniotome and multifunction handpiece

GB941R	ELAN 4 fixed dura guard pediatric	GB945R	ELAN 4 holding sleeve
GB942R	ELAN 4 fixed dura guard standard	GB947R	ELAN 4 turnable dura guard standard
GB943R	ELAN 4 fixed dura guard long		

AESCULAP® ELAN 4

PRECISION. PERFORMANCE. PLEASURE.

Engineered in Germany.

ORDER INFORMATION

› Shafts for MIS handpiece

GB920R	ELAN 4 MIS handpiece shaft, straight L10	GB925R	ELAN 4 MIS handpiece shaft, straight L13
GB921R	ELAN 4 MIS handpiece shaft, curved L10	GB926R	ELAN 4 MIS handpiece shaft, curved L13

› Accessories for automated irrigation

GA395SU	ELAN 4 electro disposable tube set for irrigation pump	GA265SU	ELAN 4 disposable spray nozzle for 2-ring handpiece L7
GA258SU	ELAN 4 disposable spray nozzle for saws	GA266SU	ELAN 4 disposable spray nozzle for 2-ring handpiece L10
GA259SU	ELAN 4 disposable spray nozzle for craniotome attachments	GA267SU	ELAN 4 disposable spray nozzle for 2-ring handpiece L13
GA261SU	ELAN 4 disposable spray nozzle for 1-ring handpiece L4	GA268SU	ELAN 4 disposable spray nozzle for 2-ring handpiece L17
GA262SU	ELAN 4 disposable spray nozzle for 1-ring handpiece L7	GA269SU	ELAN 4 disposable spray nozzle for 2-ring handpiece L22
GA263SU	ELAN 4 disposable spray nozzle for 1-ring handpiece L10	GB866R	ELAN 4 lateral burr guard for 1-ring handpiece and a max. burr diameter of 4.5 mm
GA264SU	ELAN 4 disposable spray nozzle for 1-ring handpiece L13		

› Reprocessing

GB053R	ELAN 4 air ECCOS holder for motor hose	GB087R	ELAN 4 electro ECCOS holder for reciprocating saws GA832, GA837
GB056R	ELAN 4 air ECCOS holder for perforator driver GA722	GB088R	ELAN 4 electro ECCOS holder for sagittal saw GA831
GB057R	ELAN 4 air ECCOS holder for low speed motor Intra GA724	GB260R	ECCOS set ELAN 4 air
GB062R	ELAN 4 air ECCOS holder for 1 ELAN 4 handpiece	GB261R	ECCOS set ELAN 4 electro
GB063R	ELAN 4 air ECCOS holder for 3 ELAN 4 handpieces	GB572R	ECCOS holder for cranial perforators GB300R-GB304R
GB064R	ELAN 4 air ECCOS holder for MIS handpiece GA760	GB600	STERILIT Power Systems oil spray
GB065R	ELAN 4 air ECCOS holder for craniotome GA749	GB679R	ELAN 4 rinsing device for 4 MIS shafts
GB066R	ELAN 4 air ECCOS holder for micro sagittal saw GA736	GB691R	ELAN 4 air rinsing device
GB067R	ELAN 4 air ECCOS holder for reciprocating saws GA732, GA737	GB692R	ELAN 4 electro rinsing device
GB068R	ELAN 4 air ECCOS holder for sagittal saw GA731	GB693	Ceramic filter for rinsing device
GB072R	ECCOS cable/hose positioning guide	GB718R	ELAN 4 ECCOS holder for 12 burs
GB073R	ELAN 4 electro ECCOS holder for motor cable	GB719R	ELAN 4 ECCOS holder for 3 craniotome attachments
GB076R	ELAN 4 electro ECCOS holder for perforator driver GA822	GB720R	ECCOS universal holder for saw blades
GB077R	ELAN 4 electro ECCOS holder for low speed motor Intra GA824	GB723R	ECCOS holder for 2 MIS shafts
GB082R	ELAN 4 electro ECCOS holder for 1 ELAN 4 handpiece	GB729R	ECCOS 2 for reciprocating saw shafts
GB083R	ELAN 4 electro ECCOS holder for 3 ELAN 4 handpieces	GB600850	ELAN 4 air oil spray adaptor
GB084R	ELAN 4 electro ECCOS holder for MIS handpiece GA860	GB600860	ELAN 4 electro oil spray adaptor
GB085R	ELAN 4 electro ECCOS holder for craniotome GA849	GB600870	ELAN 4 MIS oil spray adaptor
GB086R	ELAN 4 electro ECCOS holder for micro sagittal saw GA836		

AESCULAP[®] – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 7461 95-0 | Fax +49 7461 95-2600 | www.aesculap.com

Manufacturer acc. to MDD 93/42/EEC

The main product trademark "Aesculap" is a registered trademark of Aesculap AG.
Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.